

THANK YOU FOR ORDERING

Please follow the instructions for proper handling, storage prior to use, and for reporting to Organogenesis, if required.



1. OPEN BOX AND EVALUATE APLIGRAF®

- Remove the polybag from the shipping box.
- Turn over the polybag and check the pH of the **agarose gel** using the affixed color-coded pH chart (see picture on the right).
DO NOT open or use the product if it is out of pH range.
- Examine for evidence of damage.
DO NOT open or use Apligraf if there is any evidence of damage to product or polybag.
DO NOT staple any paperwork to the polybag.
- Prior to opening the polybag check the expiration date on the label.
DO NOT open or use the product if it is expired.
- Apligraf **MUST BE USED** within 15 MINUTES of opening the polybag.
DO NOT open the polybag unless it is intended to be used by the physician.



Note: Some condensation from the agarose gel medium inside the polybag is normal and is not a cause for concern.

Call 1-888-HEAL-2-DAY to arrange for a return or replacement.

2. STORE UNTIL USE

- **DO NOT REFRIGERATE.**
- Keep the package insert with the polybag.
- Keep Apligraf in its shipping container (closed) under controlled temperature 68°F to 73°F (20°C to 23°C) until ready for application.

Note: The closed shipping container is the **best place** to store Apligraf.



3. REPORT TO ORGANOGENESIS

To report an adverse event or complaint to Organogenesis, or for product orders, technical support, product questions, reimbursement information please call 1-888-HEAL-2-DAY or 1-888-432-5232 and select the appropriate option.

To return records of Apligraf usage and/or recipient information to Organogenesis after product use:

- Send sealed Apligraf usage record(s) to the attention of Customer Service Dept. Organogenesis Inc., 85 Dan Road, Canton, MA 02021 or by fax to 1-781-401-1287.
- Organogenesis can provide forms with pre-addressed envelopes that can be used to record and return product usage information. To request forms please call 1-888-HEAL-2-DAY or contact your Tissue Regeneration Specialist (TRS).

Note: In accordance with requirements for tracking in 21 CFR 1271.290, Organogenesis uses a unique number for each lot of Apligraf for product tracking purposes.

Visit www.apligraf.com for additional information about Apligraf®

Manufactured and distributed by:

Organogenesis inc.
Advancing Healing

3/2021 300-153-9

APLIGRAF® is a registered trademark of Novartis AG